


Transfer from a standing position

1




Activate the brakes on the Alenti bath chair.

2

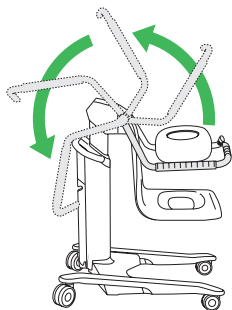

Fold back the hand rest.

3

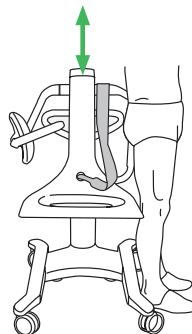


Adjust the height of the Alenti bath chair. Make sure the seat is levelled with the back of the patient's knees. Help the patient to sit down.

4




Risk of falling

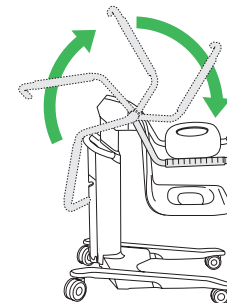

Fold forward the hand rest.

5

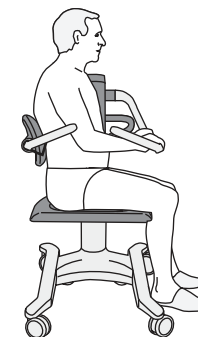

Make sure the patient is in an upright position in the centre of the seat.


6



Make sure the patient sits with the:

- buttocks at the back of the seat,
- back against the backrest and
- hands on the hand rest.

7

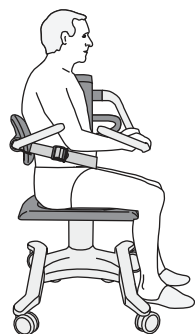

Secure the patient with the safety belt.

8



Risk of falling



Adjust the Alenti bath chair to a safe and ergonomic height.

9



Release the brakes.

10



Use the transfer handles to transfer the patient.

Transfer to a standing position

1



Lower the Alenti bath chair until the patient's feet are in contact with the floor.

2




Activate the brakes on the bath chair.

3



Fold back the hand rest away from the patient.

4

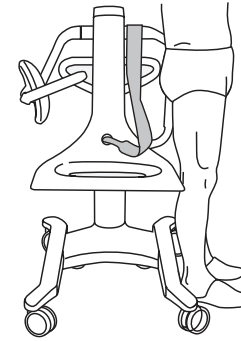

Release the safety belt, put the belt around the upper part of the hand rest-handle arm and help the patient to stand up.

Transfer from a standing and raising aid

1



Fold back the hand rest.

2




With the patient's back towards the Alenti bath chair seat, position the standing and raising aid as close as possible to the bath chair.

3




Activate the brakes on the bath chair.

4

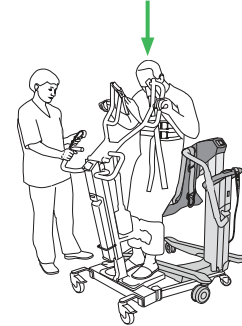


Lower the patient down on the Alenti bath chair seat. Unhook the sling.

5

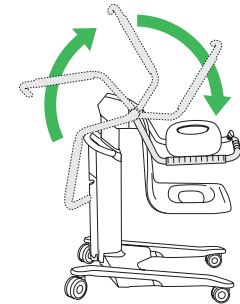

Fold forward the hand rest for the patient to hold on to.

6

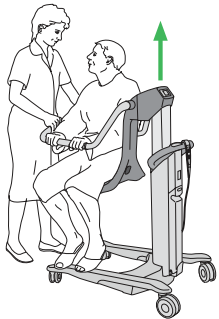

Raise the Alenti bath chair until the patient's feet are free from the standing and raising aid. Remove the sling from the patient.

7



Risk of falling




Make sure the patient sits with the:

- buttocks at the back of the seat,
- back against the backrest and
- hands on the hand rest.

8



Risk of falling

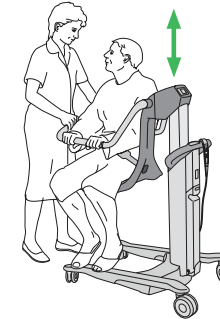

Secure the patient with the safety belt. Remove the standing and raising aid. Adjust the Alenti bath chair to an ergonomic height.


9



Release the brakes.

Transfer to a standing and raising aid


- 

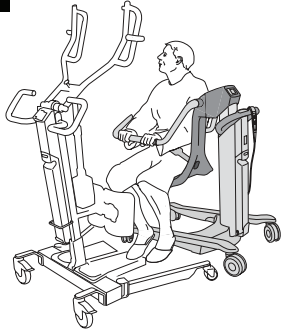
Lower the Alenti bath chair.
- 


Position the standing and raising aid in front of the patient.
- 

Leave the Alenti bath chair brakes unlocked.
- 

Apply brakes

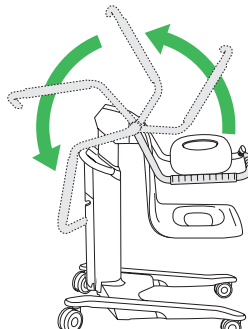Apply brakes on the standing and raising aid.
- 

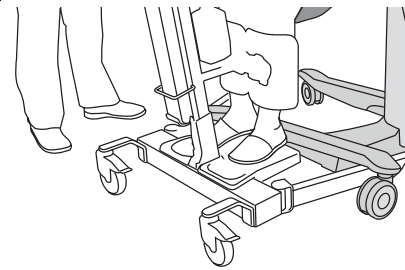Fold back the hand rest away from the patient.
- 

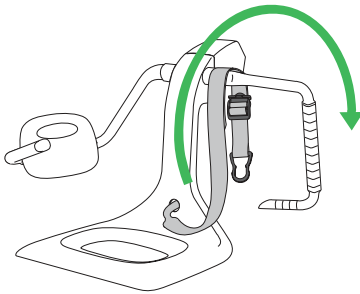Position the patient's feet onto the standing and raising aid foot support.
- 

Take off the safety belt and put it around the upper part of the hand rest.
- 

Put the sling around the patient. Attach the sling to the standing and raising aid. Gently raise the standing and raising aid.
- 

Release the brakes of the standing and raising aid and slowly move it and the patient away from the Alenti bath chair.

Transfer from bed - sitting

1




Apply brakes



Apply brakes on the bed.
Fold back the hand rest.

2

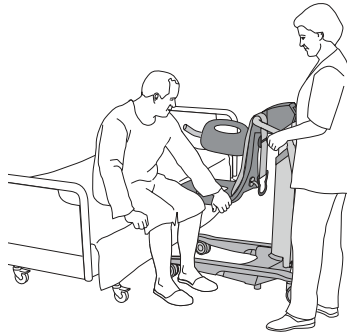


Position the Alenti bath chair front towards the side of the bed with the Alenti bath chair seat lower than the bed.

3




Activate the brakes on the Alenti bath chair.

4

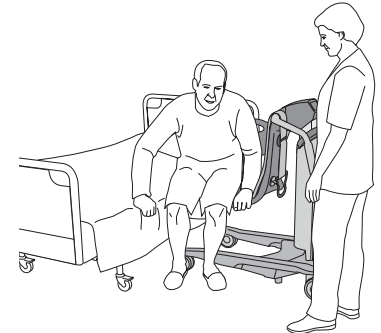

Let the patient move himself or herself from bed to the Alenti bath chair.

5

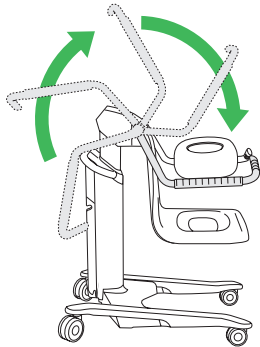

Fold forward the hand rest for the patient to hold on to.

6



Risk of falling




Make sure the patient sits with the:

- buttocks at the back of the seat,
- back against the backrest and
- hands on the hand rest.

7



Risk of falling

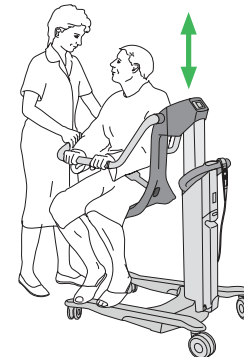

Secure the patient with the safety belt.
Adjust the Alenti bath chair to an ergonomic height.


8



Release the brakes.

Transfer to bed - sitting

1

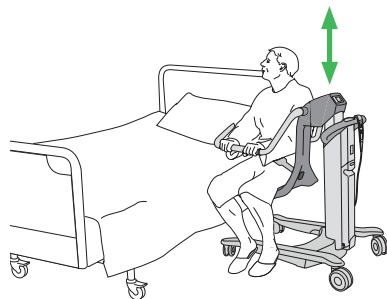

Adjust the Alenti bath chair to an appropriate height. Position the Alenti bath chair with its front facing the side of the bed.


2



Apply brakes


Activate the brakes on the Alenti bath chair.
Apply brakes on the bed.

3

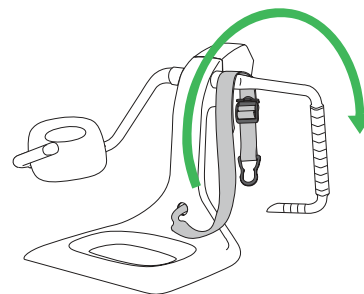


Take off the safety belt and put it around the upper part of the hand rest-handle arm.

4

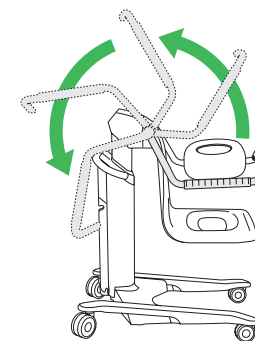


Fold back the hand rest away from the patient.

5

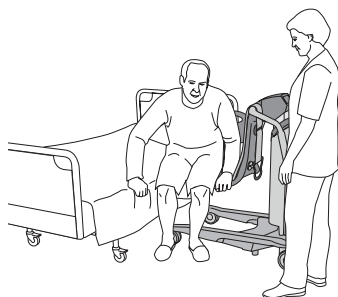

Let the patient move himself or herself to the bed from the Alenti bath chair.


6

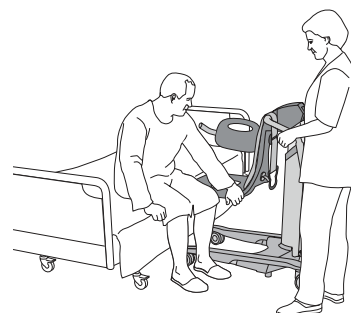

Make sure the patient sits upright securely at the side of the bed before the Alenti bath chair is pulled away.


Bathe the patient

1

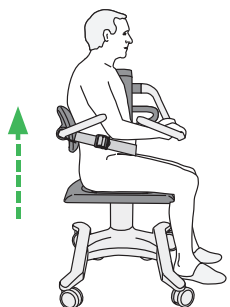

Lower the bathtub to its lowest height. Raise the bath chair so there is space between patient's genitals and the bathtub rim.

2

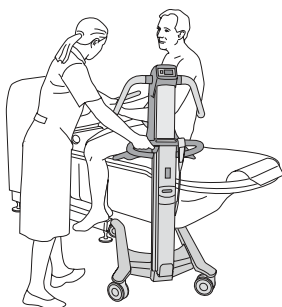


Position the bath chair diagonally towards the head end of the bathtub.

3




Risk of pinching

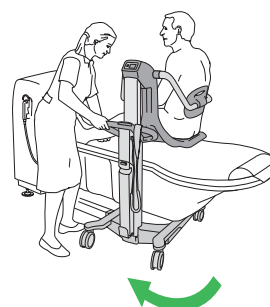


Help the patient to move their legs over the bathtub rim while rotating the bath chair into position.

4

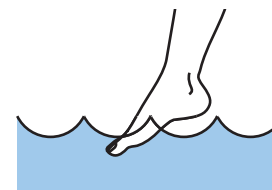

Let the patient feel the water. Adjust temperature if needed.

5

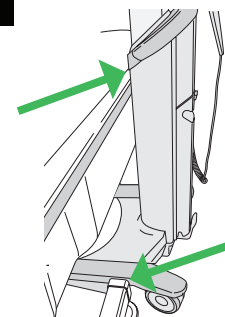

Centre the bath chair over the bathtub until it stops against the bathtub.

6





Risk of pinching



Lower the bath chair to a comfortable height for the caregiver. Raise the bathtub. Make sure no body parts get pinched.

7

Optional

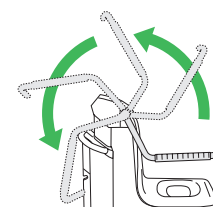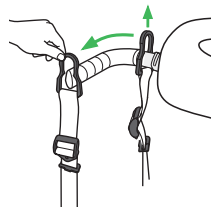

If the patient desires and is capable:

1. Take off the safety belt from the backrest.
2. Lean the patient forward.
3. Fold back the backrest.
4. Lean the patient back against the bathtub.
5. Fold back the hand rest away from the patient.


8



Bathe the patient.

After the bath

1

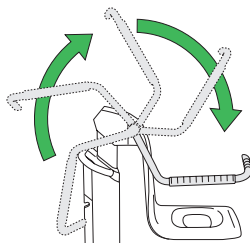


Open the drain.
If the hand rest is folded away, fold it forward in front of the patient.

2




Risk of falling

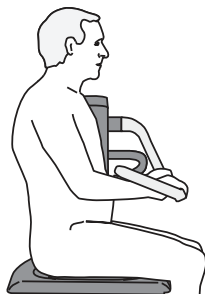

Make sure the patient sits with the:


- buttocks at the back of the seat,
- back against the backrest and
- hands on the hand rest.

3

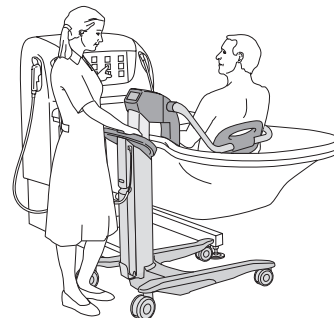

Lean the patient forward and rotate forward the backrest-handle arm.
Fasten the safety belt.

4

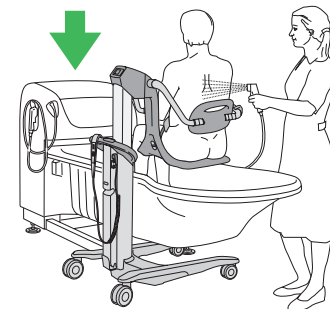

Lower the bathtub to its lowest position.
Start the shower and let the patient feel the water by hand. Adjust temperature if needed. Shower the patient.

5



Dry the patient and place towels to keep him or her warm and comfortable.

6



Make sure the patient is in an upright position in the centre of the seat.

7



Raise the bath chair so there is space between patient's genitals and the bathtub rim.
Shower and dry the patient's feet.

8



Risk of pinching



Help the patient to move their legs over the bathtub rim while rotating the bath chair away from the bathtub.

9



Risk of tipping



Make sure the Alenti bath chair is completely clear from surrounding obstacles.
Lower the bath chair.