

SKILLS CHECKLIST

Alenti

Performance observation: Employee completed equipment training and is able to demonstrate correct usage as stated in the Instructions for Use (IFU) manual.

Employee Name (Print) _____ Date _____

Department _____

All questions must be answered correctly. Fill in the correct answers below.

1.	What is the maximum weight capacity of the Alenti®?	☐ 350 lbs	☒ 400 lbs	☐ 440 lbs
2.	The Alenti can transfer someone from the bed (if policy allows).	☒ True	☐ False	
3.	The safety belt must be used at ALL times.	☐ True	☒ False	
4.	Never leave a resident/patient unattended while on the Alenti.	☒ True	☐ False	
5.	The Alenti can be used on all residents/patients.	☐ True	☒ False	
6.	The staff should ensure the Alenti is at least 1 foot away from any object that may cause an interference before raising/lowering it and that the resident/patient cannot pull or hold onto stable objects at any time (particularly when moving the Alenti or when the brakes are applied).	☒ True	☐ False	
7.	Prior to lifting the Alenti out of the tub, the resident/patient should be assessed to see if they need to be repositioned.	☒ True	☐ False	
8.	The staff should ensure that they remove the resident/patient from the bath by completely lowering the tub & raising the Alenti only to the extent necessary to clear the tub.	☒ True	☐ False	
9.	In order to use the Alenti, the resident/patient needs trunk control & the ability to sit in an upright position without support.	☒ True	☐ False	

If performance is not met, additional training is required.

	Met	Not Met
1. Explain your assessment criteria to the coach.	☐	☐
2. Demonstrate how to correctly apply the safety belt.	☐	☐
3. Identify the following parts on the Alenti: mast and remote controls, brakes, back rest, battery.	☐	☐
4. Demonstrate how to maneuver the Alenti with the coach in it.	☐	☐
5. Demonstrate how to change the batteries and the charging procedure.	☐	☐
6. Demonstrate how to transfer the coach from the bed onto the Alenti (if applicable).	☐	☐
7. Explain how to clean and disinfect the Alenti following a bath.	☐	☐

Employee Signature _____

Evaluator Signature _____