ResMed

AirSense™ 10

ENGLISH

This AirSense 10 device (model 37496) is not equipped with a wireless module.

This addendum replaces content found in the User Guide and Clinical Guide.

User guide

The following information replaces content found in the User Guide.

Welcome

Note for blind or visually impaired users – an electronic version of these instructions for use is available online on the product information pages at ResMed.com.

At a glance

Your AirSense 10 also includes the following:

- 2-pack hypoallergenic filter

About the control panel

The following icons are not applicable to your device.

Wireless signal strength (green)

Wireless transfer not enabled (grey)

No wireless connection

Airplane mode

Cleaning

Note: Disinfection is not required when used by a single patient in the home environment.

Therapy data

Your AirSense 10 device records your therapy data for you and your care provider so they can view and make changes to your therapy if required. The data is recorded and then transferred to your care provider via an SD card.

Data transmission

Wireless data transmission is not available for your device. To transfer your therapy data to your care provider, see the **SD card** section of the User Guide.

Travelling by plane

When using the device on a plane:

- **Airplane mode** is not available or required for your device.

Troubleshooting

Problem/possible cause	Solution
My therapy data has not been sent to my care provider	
Wireless data transmission is not available for your device.	Talk to your care provider about transferring therapy data. See the SD card section of the User Guide.

Technical specifications

Wireless module None	Declaration of conformity (DoC to the Radio Equipment Directive) 
-------------------------	--

Clinical guide

The following information replaces content found in the Clinical Guide.

At a glance

The AirSense 10 also includes the following:

- 2-pack hypoallergenic filter

About the control panel

The following icons are not applicable to this device.

Wireless signal strength (green)

No wireless connection

Wireless transfer not enabled (grey)

Airplane mode

Reprocessing the humidifier and air outlet

Note: Reprocessing of the airway is possible with a validated ozone disinfection procedure. More information is available from ResMed or the product page of the disinfection device manufacturer

Data management and therapy compliance

For therapy management, the AirSense 10 device stores patient therapy data on the SD card. This data can be transferred via an SD Card Reader to ResMed's ResScan™ therapy management system. For more information on therapy management with ResScan, refer to the manual supplied with the software.

Remote monitoring

Remote monitoring is not available as this AirSense 10 device (model 37496) is not equipped with a wireless module.

Data storage

Data can be transferred to therapy management software via SD card. The different ways of transferring data are detailed in the table below.

Type of Data	Data transfer method		Sessions stored
	SD card to ResScan	SD Card to AirView (card-to-cloud)	
Summary data (compliance data)	✓	✓	365
Night profile data	✓	✓	Limited by usage and SD card storage capacity
High resolution flow and pressure data (25 Hz - every 40 ms)	✓		

Therapy data

Wireless data transmission is not available as this AirSense 10 device is not equipped with a wireless module. To transfer the patient's therapy data, see the **SD card** section of the User Guide.

Travelling by plane

When using the device on a plane:

- **Airplane mode** is not available or required for this device.

Troubleshooting

Problem/possible cause	Solution
The patient's therapy data has not been transmitted	
Wireless data transmission is not available for this device.	Talk to your patient about transferring therapy data. See the SD card section of the User Guide.

Technical specifications

Wireless module None	Declaration of conformity (DoC to the Radio Equipment Directive)  None
-------------------------	--

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See ResMed.com for other ResMed locations worldwide. AirSense, AirView and ResScan is a trademark and/or registered trademark of the ResMed family of companies. For patent and other intellectual property information, see ResMed.com/ip. © 2022 ResMed. 3781029/1 2022-01
ResMed Pty Ltd